

Biologix Transport and Preservation Medium

【Product name】

Transport and Preservation Medium

【Package】

Description	Cat.#	Pack size
Transport and Preservation Medium	19-1301-(Classic)	3 mL/pc×50pc/box
	19-1302-(Inactivated)	3 mL/pc×50pc/box
	19-1262-(Inactivated)	2 mL/pc×50pc/box

【Application】

For the transfer and storage of clinically collected samples

【Main component】

Including 1 swab and 1 tube containing medium.

Classic medium: HBSS, antibiotics, BSA, cryoprotectant, biological buffer, phenol red.

Inactivated medium: Guanidine salt (denaturant), reducing agent, biological buffer, chelating agent, surfactant, buffer.


【Storage conditions and validity】

12 months storage at 5-25 °C


【Sample requirements】

1. Applicable sample: Clinical samples such as oral swab, nasopharyngeal swab, saliva, sputum;
2. Storage and transportation of samples: Please pay attention to the storage period when using. Specimens should be transported to the corresponding laboratory within 2 working days after collection, and the storage temperature is 2-8 °C; If it cannot be delivered to the laboratory within 48 hours, it should be stored at -80 °C or below, and ensure that the collected specimens are delivered to the corresponding laboratory within 1 week. Specimens should avoid repeated freeze-thaw cycles.


【Usage】

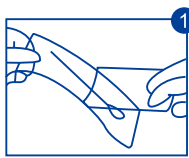


Unwrap the package, take out the swab.

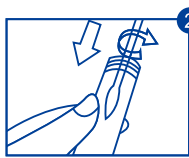

Use the swab to collect samples and put it back into the tube.

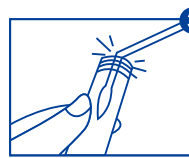

Break the swab rod and leave the sampling parts inside the tube.

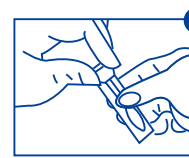

Screw the lid tightly, and mark on the tube.

【Notice】

1. Used for in vitro diagnosis, not for internal or external use in humans or animals. It can cause serious events if swallowed; it is irritating to eyes and skin. If accidentally splashed into eyes, rinse with water. Ventilation should be maintained during use.
2. This product is red or colorless, transparent, non-precipitating liquid. If the properties change, such as yellow, it is prohibited to use.

BIOLGIX Biologix Corporation

No.2766 Ying Xiu Road
High-Tech Industrial Development Zone
Jinan, Shandong 250101
P.R. China.

Tel:86-531-67802668
Email:Inquiry@BiologixGroup.com
Web:www.BiologixGroup.com